# wellion® **FRIGO**

SE

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**Wellion® FRIGO** is the easy, quick and comfortable way to cool your insulin, insulin pump and other medicine.  
ANTIBACTERIAL - USE WITHOUT POWER SUPPLY - USE WITHOUT FREEZER

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# wellion® FRIGO

EN

**Wellion® FRIGO** is the easy, quick and comfortable way to cool your insulin, an insulin pump and other medicine. For this neither electricity nor freezing or refrigeration is necessary. **Wellion® FRIGO** can be used **very simply** with the aid of **water** - over and over again.

To ensure constant quality and hygiene at frequent use, the inner pocket is featured with an antibacterial impregnation - an additional feature which is very important in the handling of insulin and medicines.

#### Contents:

Each package consists of:

1. **Wellion® FRIGO** Outer pocket
2. **Wellion® FRIGO** Inner pocket filled with crystals

#### User guide:

1. Shake the **Wellion® FRIGO** Inner pocket to disperse the crystals evenly.
2. Immerse the **Wellion® FRIGO** Inner pocket completely in cold water (**Wellion® FRIGO** L maximum 10 minutes, **Wellion® FRIGO** XXL maximum 12 minutes). Do not immerse the inner pocket longer in water as specified. **A small discharge of gel may occur, wipe off the gel and use the pocket as intended.**
3. The crystals in the cooling ribs will build up a gel, the inner pocket will grow thicker.
4. After that it is necessary to wipe off the inner pocket to remove surplus water.
5. Put the cooled insulin into the **Wellion® FRIGO** inner pocket and put the inner pocket into the **Wellion® FRIGO** outer pocket.

#### Permanent use and storage:

Immerse the **Wellion® FRIGO** inner pocket only in water, if the gel has already reformed into crystals. While permanent use of the pocket, the immersion period is shorter according to the degree of humidity of the crystals. Take the inner pocket out of the water as soon as it's well-stocked again. **A small discharge of gel may occur, wipe off the gel and use the pocket as intended.**

Do not store your active **Wellion® FRIGO** longer than 24 hours in an air tight bin (backpack, suitcase, etc.). Because of the principle of evaporation the pocket requires circulation of air.

While non-use of the pocket, separate the inner pocket from the outer pocket, to allow drying out of the inner pocket. Shake the pocket while drying to avoid sticking together of the crystals.

MED TRUST Handelsgesellschaft m.b.H.  
Gewerbepark 10 • A-7221 Marz  
Tel.: (+43)02626/64190 • e-mail: office@medtrust.at • www.wellion.at

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MED TRUST GmbH • Zur Wetterwarte 50/Haus 337L • D-01109 Dresden