

DEVIATION TABLE WITH +/- 20%

Calibration of your CGMS is optional. You can manually input the blood glucose meter value for calibration when the difference to the sensor value is greater than 20%. It's recommended to use blood glucose meter readings, which are more than 72 mg/dL to calibrate your CGMS.

- Calibration with your blood glucose meter may align your CGMS readings with your blood glucose values.
- If you want to calibrate your CGMS using your blood glucose meter, enter the blood glucose meter value within 5 minutes of taking a fingerstick.
- Calibration is not recommended when glucose values are rising or falling fast.



-20%	Glucose	+20%	-20%	Glucose	+20%	-20%	Glucose	+20%
29	36	43	61	76	91	93	116	139
30	37	44	62	77	92	94	117	140
30	38	46	62	78	94	94	118	142
31	39	47	63	79	95	95	119	143
32	40	48	64	80	96	96	120	144
33	41	49	65	81	97	97	121	145
34	42	50	66	82	98	98	122	146
34	43	52	66	83	100	98	123	148
35	44	53	67	84	101	99	124	149
36	45	54	68	85	102	100	125	150
37	46	55	69	86	103	101	126	151
38	47	56	70	87	104	102	127	152
38	48	58	70	88	106	102	128	154
39	49	59	71	89	107	103	129	155
40	50	60	72	90	108	104	130	156
41	51	61	73	91	109	105	131	157
42	52	62	74	92	110	106	132	158
42	53	64	74	93	112	106	133	160
43	54	65	75	94	113	107	134	161
44	55	66	76	95	114	108	135	162
45	56	67	77	96	115	109	136	163
46	57	68	78	97	116	110	137	164
46	58	70	78	98	118	110	138	166
47	59	71	79	99	119	111	139	167
48	60	72	80	100	120	112	140	168
49	61	73	81	101	121	113	141	169
50	62	74	82	102	122	114	142	170
50	63	76	82	103	124	114	143	172
51	64	77	83	104	125	115	144	173
52	65	78	84	105	126	116	145	174
53	66	79	85	106	127	117	146	175
54	67	80	86	107	128	118	147	176
54	68	82	86	108	130	118	148	178
55	69	83	87	109	131	119	149	179
56	70	84	88	110	132	120	150	180
57	71	85	89	111	133	121	151	181
58	72	86	90	112	134	122	152	182
58	73	88	90	113	136	122	153	184
59	74	89	91	114	137	123	154	185
60	75	90	92	115	138	124	155	186

DEVIATION TABLE WITH +/- 20%

Calibration of your CGMS is optional. You can manually input the blood glucose meter value for calibration when the difference to the sensor value is greater than 20%. It's recommended to use blood glucose meter readings, which are more than 72 mg/dL to calibrate your CGMS.

- Calibration with your blood glucose meter may align your CGMS readings with your blood glucose values.
- If you want to calibrate your CGMS using your blood glucose meter, enter the blood glucose meter value within 5 minutes of taking a fingerstick.
- Calibration is not recommended when glucose values are rising or falling fast.




-20%	Glucose	+20%	-20%	Glucose	+20%	-20%	Glucose	+20%
125	156	187	157	196	235	189	236	283
126	157	188	158	197	236	190	237	284
126	158	190	158	198	238	190	238	286
127	159	191	159	199	239	191	239	287
128	160	192	160	200	240	192	240	288
129	161	193	161	201	241	193	241	289
130	162	194	162	202	242	194	242	290
130	163	196	162	203	244	194	243	292
131	164	197	163	204	245	195	244	293
132	165	198	164	205	246	196	245	294
133	166	199	165	206	247	197	246	295
134	167	200	166	207	248	198	247	296
134	168	202	166	208	250	198	248	298
135	169	203	167	209	251	199	249	299
136	170	204	168	210	252	200	250	300
137	171	205	169	211	253	201	251	301
138	172	206	170	212	254	202	252	302
138	173	208	170	213	256	202	253	304
139	174	209	171	214	257	203	254	305
140	175	210	172	215	258	204	255	306
141	176	211	173	216	259	205	256	307
142	177	212	174	217	260	206	257	308
142	178	214	174	218	262	206	258	310
143	179	215	175	219	263	207	259	311
144	180	216	176	220	264	208	260	312
145	181	217	177	221	265	209	261	313
146	182	218	178	222	266	210	262	314
146	183	220	178	223	268	210	263	316
147	184	221	179	224	269	211	264	317
148	185	222	180	225	270	212	265	318
149	186	223	181	226	271	213	266	319
150	187	224	182	227	272	214	267	320
150	188	226	182	228	274	214	268	322
151	189	227	183	229	275	215	269	323
152	190	228	184	230	276	216	270	324
153	191	229	185	231	277	217	271	325
154	192	230	186	232	278	218	272	326
154	193	232	186	233	280	218	273	328
155	194	233	187	234	281	219	274	329
156	195	234	188	235	282	220	275	330

DEVIATION TABLE WITH +/- 20%

Calibration of your CGMS is optional. You can manually input the blood glucose meter value for calibration when the difference to the sensor value is greater than 20%. It's recommended to use blood glucose meter readings, which are more than 72 mg/dL to calibrate your CGMS.

- Calibration with your blood glucose meter may align your CGMS readings with your blood glucose values.
- If you want to calibrate your CGMS using your blood glucose meter, enter the blood glucose meter value within 5 minutes of taking a fingerstick.
- Calibration is not recommended when glucose values are rising or falling fast.

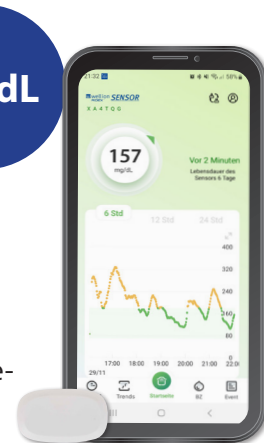


-20%	Glucose	+20%	-20%	Glucose	+20%	-20%	Glucose	+20%
221	276	331	253	316	379	285	356	427
222	277	332	254	317	380	286	357	428
222	278	334	254	318	382	286	358	430
223	279	335	255	319	383	287	359	431
224	280	336	256	320	384	288	360	432
225	281	337	257	321	385	289	361	433
226	282	338	258	322	386	290	362	434
226	283	340	258	323	388	290	363	436
227	284	341	259	324	389	291	364	437
228	285	342	260	325	390	292	365	438
229	286	343	261	326	391	293	366	439
230	287	344	262	327	392	294	367	440
230	288	346	262	328	394	294	368	442
231	289	347	263	329	395	295	369	443
232	290	348	264	330	396	296	370	444
233	291	349	265	331	397	297	371	445
234	292	350	266	332	398	298	372	446
234	293	352	266	333	400	298	373	448
235	294	353	267	334	401	299	374	449
236	295	354	268	335	402	300	375	450
237	296	355	269	336	403	301	376	451
238	297	356	270	337	404	302	377	452
238	298	358	270	338	406	302	378	454
239	299	359	271	339	407	303	379	455
240	300	360	272	340	408	304	380	456
241	301	361	273	341	409	305	381	457
242	302	362	274	342	410	306	382	458
242	303	364	274	343	412	306	383	460
243	304	365	275	344	413	307	384	461
244	305	366	276	345	414	308	385	462
245	306	367	277	346	415	309	386	463
246	307	368	278	347	416	310	387	464
246	308	370	278	348	418	310	388	466
247	309	371	279	349	419	311	389	467
248	310	372	280	350	420	312	390	468
249	311	373	281	351	421	313	391	469
250	312	374	282	352	422	314	392	470
250	313	376	282	353	424	314	393	472
251	314	377	283	354	425	315	394	473
252	315	378	284	355	426	316	395	474

DEVIATION TABLE WITH +/- 20%

Calibration of your CGMS is optional. You can manually input the blood glucose meter value for calibration when the difference to the sensor value is greater than 20%. It's recommended to use blood glucose meter readings, which are more than 72 mg/dL to calibrate your CGMS.

- Calibration with your blood glucose meter may align your CGMS readings with your blood glucose values.
- If you want to calibrate your CGMS using your blood glucose meter, enter the blood glucose meter value within 5 minutes of taking a fingerstick.
- Calibration is not recommended when glucose values are rising or falling fast.

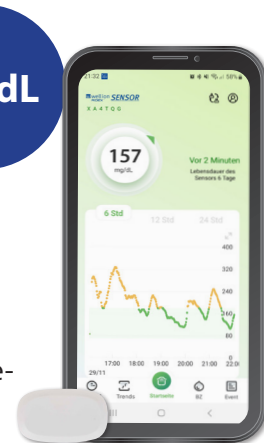

-20%	Glucose	+20%	-20%	Glucose	+20%
317	396	475	349	436	523
318	397	476	350	437	524
318	398	478	350	438	526
319	399	479	351	439	527
320	400	480	352	440	528
321	401	481	353	441	529
322	402	482	354	442	530
322	403	484	354	443	532
323	404	485	355	444	533
324	405	486	356	445	534
325	406	487	357	446	535
326	407	488	358	447	536
326	408	490	358	448	538
327	409	491	359	449	539
328	410	492	360	450	540
329	411	493			
330	412	494			
330	413	496			
331	414	497			
332	415	498			
333	416	499			
334	417	500			
334	418	502			
335	419	503			
336	420	504			
337	421	505			
338	422	506			
338	423	508			
339	424	509			
340	425	510			
341	426	511			
342	427	512			
342	428	514			
343	429	515			
344	430	516			
345	431	517			
346	432	518			
346	433	520			
347	434	521			
348	435	522			

